

QUIK GLUE

- CD4150 White 50cc 30 second set time
- CD4220 White 220cc 30 second set time
- CD4350 Tan 50cc 60 second set time
- CD4320 Tan 220cc 60 second set time
- CD4250 Black 50cc 90 second set time
- CD4220S Black 220cc 90 second set time
- **CD401** | 6 Pack Mix Tips for 50 cc Cartridge
- CD402 | 20 Pack Extra Mix Tips for 50 cc Cartridge
- CD412 | 12 Pack Extra Mix Tips for 220 cc Cartridge

CDDMA51 | 50cc Dispenser is required for use (sold separately)

CDDM220 | 220cc Dispenser is required for use (sold separately)

Made in USA



QUIKGLUE.revA.09292021

Manufactured by Coyote®

419 N. Curtis Rd., Boise, ID 83706


(208) 429-0026 | www.coyote.us



Parts Included


Coyote Quik Glue 50cc or 220cc (1)

Small Mix Tip or Large Mix Tip (1)

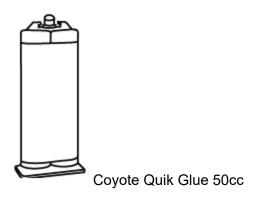



Small Mix Tip

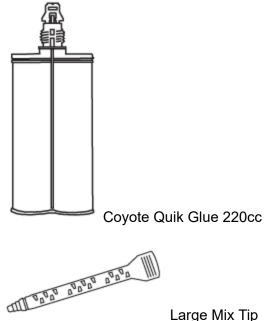

Large Mix Tip

For instructional translations go to:

Para traducciones instructivas, vaya a:

Gebrauchsanweisungen finden Sie unter:

Pour les traductions pédagogiques, rendez-vous sur:

Per le traduzioni didattiche vai a:

For instruksjonsoversettelser, gå til:

For instruktionsoversættelser, gå til:


Po tłumaczenia instruktażowe przejdź do:

www.coyote.us/instructions-quikglue

Instructions for 50cc Quik Glue




1. Purge the cartridge Snip off the tip. Insert cartridge into dispenser gun. Expel a small amount of adhesive to purge the cartridge.

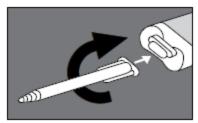

2. Install mix tip Line up mix tip appropriately and twist into place.

Tips can only be used once.

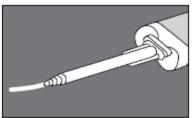

3. Run a test bead approx. 1" in length. Store unused portion by leaving the mix tip on when done.



4. Leave used tip attached as a cap until next use to keep glue

from drying out in the tubes.

When your ready to use the glue again remove old tip and replace with a new tip.


Attention

1. Read carefully BEFORE applying adhesive to ensure proper adhesion.

2. Quik Glue sets fast. To ensure a proper bond, work quickly. Refrigerate the cartridge to slow down work time.

Parts Sold Separately

CDDMA51 | Dispenser is required for use (sold separately)

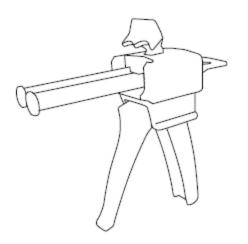


CD401 | 6 - Pack Extra Mix Tips for 50 cc Cartridge.

CD402 | 20 - Pack Extra Mix Tips for 50 cc Cartridge.




Instructions for 220cc Quik Glue

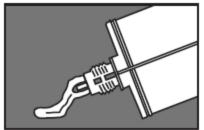


1. Purge the cartridge. Snap off the tip with a screwdriver. Insert

the cartridge into the dispenser gun. Expel a small amount of adhesive to purge the cartridge.

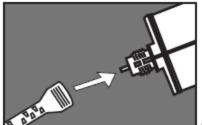


2. Install mix tip Line up mix tip appropriately and screw into

place. Tips can only be used once.

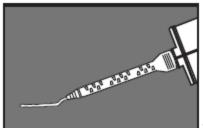

3. Run a test bead approx. 1" in length. Store unused portion by

leaving the mix tip on when done.

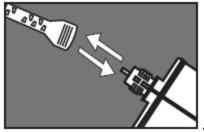

→ 4. Leave used tip attached as a cap until next use to keep glue

from drying out in the tubes. When your ready to use the glue again remove the old tip and replace it with a new tip.


Attention:

1. Read carefully BEFORE applying adhesive to ensure proper adhesion.

2. Quik Glue sets fast. To ensure a proper bond, work quickly. Refrigerate the cartridge to slow down work time.

Parts Sold Separately for the 220cc

CDDM220 | Dispenser is required for use (sold separately)

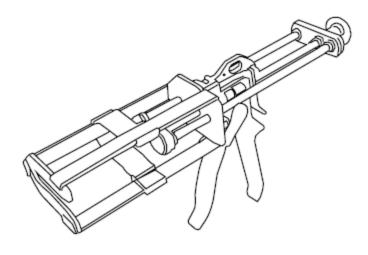

CD412 | 12-Pack Extra Mix Tips for 220 cc Cartridge.

all vavav avava avava



Maintenance

To ensure maximum performance, the fulcrum pins in the handle should be lubricated with WD-40 (or equivalent) after every 10 uses.

Always wipe off any adhesive using manufactures recommended solvent before it has time to cure.

ATTENTION

Never operate the applicator with missing or loose parts.

Remove any unwanted material residue from applicator using a soft cloth and a solvent recommended by the material supplier.

Always wear safety glasses while operating this product and follow the material suppliers usage instructions.

DISCLAIMER

While Coyote® has taken every care in the preparation of this document, it cannot accept responsibility for omissions, and does not warrant that it is correct and comprehensive in every particulate detail. Equipment supplied should always be operated by persons with an appropriate level of skill and training. Coyote® shall not be liable for incidental or consequential damages resulting from the misuse/abuse of any of its products. Manufactured in the UK to ISO 9001:2008 standards.

Every effort has been made to ensure that the information contained in this manual was correct at the time of going to press. However, the company retains the right to change any specification without notice, in line with the company's policy of continual product development.

Contact Coyote for a MSDS for this product coyote@coyoteprosthetics.com