


**PARTS SOLD SEPARATELY**

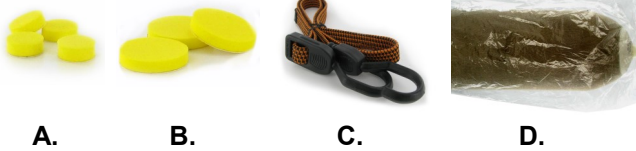

A. CD103SFC Small Foam Circles B. 103LFC Large Foam Circles  
C. CD306 Casting Bungee D. CD308 Casting Bags



# CD316SA

## Casting Handle and Pin for Small Air-Lock

- |                                |                                  |
|--------------------------------|----------------------------------|
| EN   Instructions for Use      | CS   Návod k použití             |
| DE   Gebrauchsanweisung        | TR   Kullanım Talimatları        |
| FR   Notice d'utilisation      | RU   Инструкция по использованию |
| ES   Instrucciones para el uso | JA   取扱説明書                       |
| IT   Istruzioni per l'uso      | ZH   中文说明书                       |
| NO   Bruksanvisning            | KO   사용 설명서                      |
| DA   Brugsanvisning            |                                  |
| SV   Bruksanvisning            |                                  |
| EL   Οδηγίες Χρήσης            |                                  |
| FI   Käyttöohjeet              |                                  |
| NL   Gebruiksaanwijzing        |                                  |
| PT   Instruções de Utilização  |                                  |
| PL   Instrukcja użytkowania    |                                  |



[www.coyote.us/instructions-accessories](http://www.coyote.us/instructions-accessories)



MADE IN USA

External Prosthetic Components



8 11576 02066 4



CD316SA.rev.b.10272021



Manufactured by Coyote®  
419 N. Curtis Rd., Boise, Idaho 83706  
(208) 429-0026 | [www.coyote.us](http://www.coyote.us)



### Casting handle


For easier casting and fabrication, and to prevent incorrect lock placement, we recommend using the Casting Handle. If using a distal cushion, it is best if the cushion is cast into place. In some cases, memory foam cushions can be added after the fact.



1. Install casting pin in liner of choice
2. Slide handle onto pin
3. From above and the side, make sure the liner/pin is in the desired location
6. Foam circles can be used to mark key areas (CD103SFC and CD103LFC)
7. Stockinette is put over the handle and liner



8. With first couple of wraps, go around handle to key into place, then
9. Continue to check location and angle of handle during casting to ensure the lock placement is as desired
10. Once cast is removed, install lock anchor, and fill cast
11. Handle has identical shape as lock for proper seating and location
12. Cast and anchor ready to receive new lock.

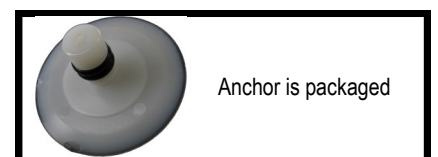

Anchor is packaged