

Parts Included



Coyote®

CD120L

Lanyard and Low Profile Screw

Fabrication Instructions



Weight limit: 265 lbs.

2-year warranty against manufacturer defects, excessive wear or breakage.

Made in U.S.A.

External Prosthetic Components



EC REP

Advena Limited
Tower Business Centre
2nd Flr, Tower Street
Swatara, BKR 4013
Malta



CD120L.revC.08122021

EN | Instructions for Use
DE | Gebrauchsanweisung
FR | Notice d'utilisation
ES | Instrucciones para el uso
IT | Istruzioni per l'uso
NO | Bruksanvisning
DA | Brugsanvisning
SV | Bruksanvisning
EL | Οδηγίες Χρήσης
FI | Käyttöohjeet
NL | Gebruiksaanwijzing
PT | Instruções de Utilização
PL | Instrukcja użytkowania

CS | Návod k použití
TR | Kullanım Talimatları
RU | Инструкция по использованию
JA | 取扱説明書
ZH | 中文说明书
KO | 사용 설명서




www.coyote.us/instructions-Lanyard-puck

Coyote®


Manufactured by Coyote®
419 N. Curtis Rd., Boise, Idaho 83706
(208) 429-0026 | www.coyote.us



Attaching Lanyard Strap

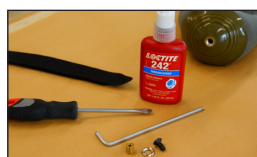

1 Attaching strap to liner.

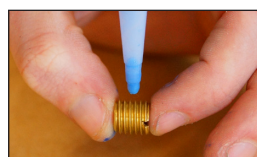


2 Use Loctite 242 on all threads.



3 Screw adaptor screw in until it is flush with the liner.



4 Use a hot awl or punch to make a hole for 6x14 screw. *Screw should fit through hole easily.*

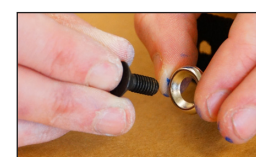

5 Put 6x14 screw and liner washer through hole in the lanyard strap.



6 Use Loctite 242 on 6x14 screw.



7 Screw in 6x14 Screw until it is tight.



8 Attach strap to liner and feed strap through lanyard slot.



9 Use lanyard strap to attach chafe in proper location.



10 Add desired rivet to hold chafe.



11 Cut strap to desired length.