

Included in package:



1 Fabrication Dummy

EN | Instructions for Use
 DE | Gebrauchsanweisung
 FR | Notice d'utilisation
 ES | Instrucciones para el uso
 IT | Istruzioni per l'uso
 NO | Bruksanvisning
 DA | Brugsanvisning
 SV | Bruksanvisning
 EL | Οδηγίες Χρήσης
 FI | Käyttöohjeet
 NL | Gebruiksaanwijzing
 PT | Instruções de Utilização
 PL | Instrukcja użytkowania

CS | Návod k použití
 TR | Kullanım Talimatları
 RU | Инструкция по использованию
 JA | 取扱説明書
 ZH | 中文说明书
 KO | 사용 설명서



www.coyote.us/instructions-accessories



Manufactured by Coyote®
 419 N. Curtis Rd., Boise, Idaho 83706
 (208) 429-0026 | www.coyote.us



CD103FD

Fabrication Dummy

For CD103 Air-Lock or CD117 Easy Off Lock
 CD103D Deep Air-Lock or CD117D Easy Off Deep lock



2-Year warranty against manufacturer defects, excessive wear or breakage

Made in U.S.A.

External Prosthetic Components



EC REP



CD103FD.revA.04162021

? b t c l _ J g k g c b
 R m u c p @ s q j c q q A c l r p c
 O l b D j p * R m u c p Q r p c c r
 Q u _ r _ p * @ l P 2 . / 1
 K _ j r _

Included in package:



1 Fabrication Dummy

EN | Instructions for Use
 DE | Gebrauchsanweisung
 FR | Notice d'utilisation
 ES | Instrucciones para el uso
 IT | Istruzioni per l'uso
 NO | Bruksanvisning
 DA | Brugsanvisning
 SV | Bruksanvisning
 EL | Οδηγίες Χρήσης
 FI | Käyttöohjeet
 NL | Gebruiksaanwijzing
 PT | Instruções de Utilização
 PL | Instrukcja użytkowania

CS | Návod k použití
 TR | Kullanım Talimatları
 RU | Инструкция по использованию
 JA | 取扱説明書
 ZH | 中文说明书
 KO | 사용 설명서



www.coyote.us/instructions-accessories



Manufactured by Coyote®
 419 N. Curtis Rd., Boise, Idaho 83706
 (208) 429-0026 | www.coyote.us



CD103FD

Fabrication Dummy

For CD103 Air-Lock or CD117 Easy Off Lock
 CD103D Deep Air-Lock or CD117D Easy Off Deep lock



2-Year warranty against manufacturer defects, excessive wear or breakage

Made in U.S.A.

External Prosthetic Components



EC REP



CD103FD.revA.04162021

? b t c l _ J g k g c b
 R m u c p @ s q j c q q A c l r p c
 O l b D j p * R m u c p Q r p c c r
 Q u _ r _ p * @ l P 2 . / 1
 K _ j r _

Flexible Inner Socket with Coyote Designs Fabrication Dummy CD103FD



1. Put CD103FD fabrication dummy over anchor.



2. Pull preferred flexible plastic over socket and installed fabrication dummy using a vacuum system.



3. Pressing in around fabrication dummy will help remove any air pockets.



4. If needed use parachute cord or rope to pull plastic tight to dummies edge.



5. Let plastic cool before cutting fabrication dummy free.



6. Cut tooling free with cast saw, knife or utility knife.



7. Remove tooling from anchor.



8. Sand flexible inner down so anchor is almost 100% exposed



9. Pull a PVA bag over the anchor and cast. Glue your lock to anchor and start laminating.

Flexible Inner Socket with Coyote Designs Fabrication Dummy CD103FD



1. Put CD103FD fabrication dummy over anchor.



2. Pull preferred flexible plastic over socket and installed fabrication dummy using a vacuum system.



3. Pressing in around fabrication dummy will help remove any air pockets.



4. If needed use parachute cord or rope to pull plastic tight to dummies edge.



5. Let plastic cool before cutting fabrication dummy free.



6. Cut tooling free with cast saw, knife or utility knife.



7. Remove tooling from anchor.



8. Sand flexible inner down so anchor is almost 100% exposed



9. Pull a PVA bag over the anchor and cast. Glue your lock to anchor and start laminating.