
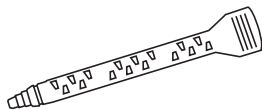


## Parts Included

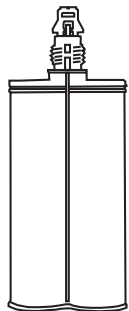

Large Mix Tip (1)

### Tech Tip -

Store all glue in cool environment for longevity.

Coyote Quik Glue 220cc

EN | Instructions for Use

DE | Gebrauchsanweisung

FR | Notice d'utilisation

ES | Instrucciones para el uso

IT | Istruzioni per l'uso

NO | Bruksanvisning

DA | Brugsanvisning

SV | Bruksanvisning

EL | Οδηγίες Χρήσης

FI | Käyttöohjeet

NL | Gebruiksaanwijzing

PT | Instruções de Utilização

PL | Instrukcja użytkowania

CS | Návod k použití

TR | Kullanım Talimatları

RU | Инструкция по использованию

JA | 取扱説明書

ZH | 中文说明书

KO | 사용 설명서



[www.coyote.us/instructions-quickglue](http://www.coyote.us/instructions-quickglue)

Manufactured by Coyote®

419 N. Curtis Rd., Boise, Idaho 83706

(208) 429-0026 | [www.coyote.us](http://www.coyote.us)



**Coyote**®

# QUIK GLUE

## TAN - CD4320

60 second set time - 220cc



Made in U.S.A.




8 11576 02236 1




CD4320.revD.02082023

**Coyote**®


## Instructions

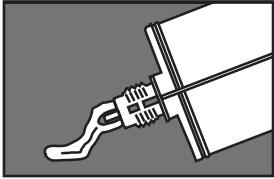


- 1** Purge the cartridge. Snap off the tip with screw driver. Insert cartridge into dispenser gun. Expel a small amount of adhesive to purge the cartridge.

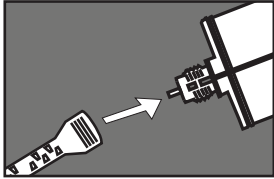

- 2** Install mix tip. Line up mix tip appropriately and screw into place. Tips can only be used once.

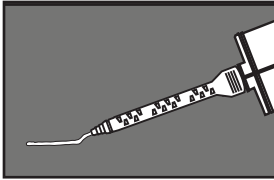

- 3** Run a test bead approx. 1" in length. Store unused portion by leaving the mix tip on when done.

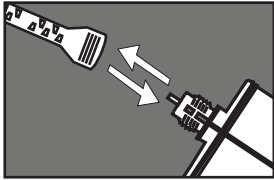


- 4** Leave used tip attached as a cap until next use to keep glue from drying out in the tubes. When your ready to use the glue again remove old tip and replace with a new tip.

## CAUTION


1. Read carefully BEFORE applying adhesive to ensure proper adhesion.
2. Quik Glue sets fast. To ensure a proper bond, work quickly. Refrigerate the cartridge to slow down work time.
3. Smaller bead size may affect cure time.

## Parts Sold Separately

**CDDM220** | Dispenser is required for use (sold separately)

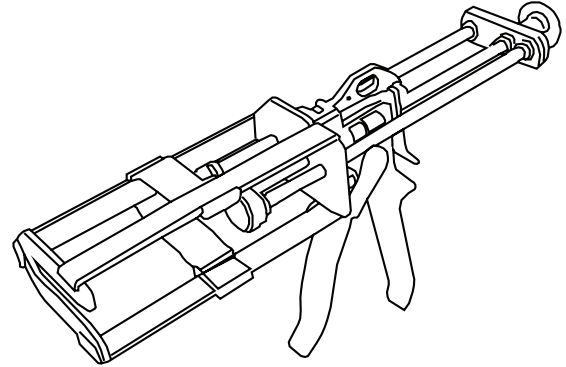

**CD412** | 12-Pack Extra Mix Tips for 220 cc Cartridge.

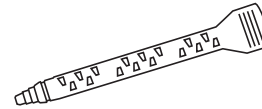

Contact Coyote for a MSDS for this product.  
(208) 429-0026 or [coyote@coyote.us](mailto:coyote@coyote.us)