


# P-54DC

## Distal Cushion Set

Distal Cushion Set for the SmartPuck™ and AirPuck™  
2/Pk, (1)-6mm, (1)-12mm

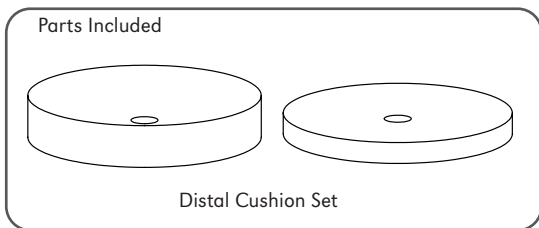

6- month warranty against manufacturer defects.

External Prosthetic Components



P-54DC.revA.01282026



Manufactured by Coyote<sup>®</sup> O&P Inc.  
525 N. Steelhead Way Boise, ID 83704 USA  
(208) 429-0026 | [www.coyote.us](http://www.coyote.us)





## ATTENTION

### Methods for Total Contact in the Distal End of the Socket

These distal cushions are ideal for test socket or demo fitting but a custom distal cushion is highly recommended for definitive sockets.


For a Custom Distal Injection Tool refer to part number AP-100DI.

Fabricate the socket to completion such that it is ready to be fit on the patient.

Coyote Distal Cushion Set (P-54DC) can be used in place of injecting a Custom Distal Cushion or Flexible Inner Liner for test sockets or definitive sockets.

The Coyote Distal Cushion set is designed to be stackable or used singularly.

It is highly recommended that you use our Distal Cushions, a Custom Distal Pad or a flexible inner liner.

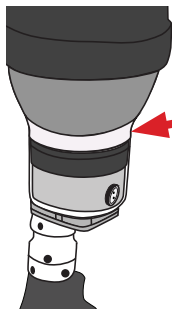

Place the Distal Cushion or pad or flexible inner liner on top of the puck and filter disk.

*With elevated vacuum fit is more important than other forms of suspension. Make sure the distal end and shape of the limb is supported with padding.*



Distal Cushion



Custom Distal Cushion



Flexible Inner

**ATTENTION:** For more in depth instructions on other options please refer to the SmartPuck/AirPuck Prosthetist Technical Guide and the Puck Fabrication Guide for Internal Elevated Vacuum Systems.

**See instructions online.**

[www.coyote.us/instructions](http://www.coyote.us/instructions)

EN | Instructions for Use  
 DE | Gebrauchsanweisung  
 FR | Notice d'utilisation  
 ES | Instrucciones para el uso  
 IT | Istruzioni per l'uso  
 NO | Bruksanvisning  
 DA | Brugsanvisning  
 SV | Bruksanvisning  
 NL | Gebruiksaanwijzing  
 RU | Инструкция по использованию

JA | 取扱説明書  
 ZH | 中文说明书  
 KO | 사용 설명서  
 FI | Käyttöohjeet  
 CS | Návod k použití  
 EL | Οδηγίες Χρήσης  
 PT | Instruções de Utilização  
 PL | Instrukcja użytkowania  
 TR | Kullanım Talimatları

