




# CD209S

## Mini Dynamic Strut

### Soft Flex Kit

Weight Range 70-110 lbs.

Light to Medium Activity Level

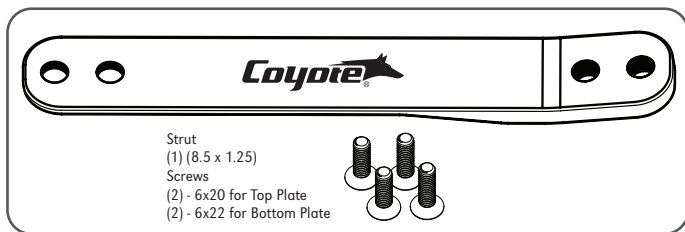

6-month warranty against manufacturer defects, excessive wear or breakage.

External Prosthetic Components



CD209S.post.revA.01182024



Manufactured by Coyote®  
419 N. Curtis Rd., Boise, Idaho 83706 USA  
(208) 429-0026 | [www.coyote.us](http://www.coyote.us)



Advena Limited  
Tower Business Centre  
2nd Flr, Tower Street  
Swatar, BKR 4013  
Malta



## **ATTENTION**

**Always, *LOCTITE*® and torque to manufacturer specifications.**

1. The instructions for this device cover a very specific method of fabrication, and we understand that a multitude of methods could be used. The fabricated device is the sole responsibility of the fabricator.
2. Always use Loctite® Blue 242 on screw threads. Torque between 5 Nm and 8 Nm in even distribution.
3. Tighten the screws after a couple weeks of use, also at follow up appointments, re-Loctite® if needed.
4. Use the 6x20mm (top mounting plate) and 6x22mm (bottom mounting plate) screws provided with typical components for the (CD209S, CD209SS, CD209M, CD209MS, CD209R, CD209RS). In atypical setups, longer screws may be needed. Always use screws class 10.9 or better.

Make sure the screws are all the way through the threaded post holes and flush with the inside of the plate. Be sure the screw length fully seats into face plate not just the post, longer screws may be needed depending on the strut and brace thickness. *See example images below.*

A 6 month limited warranty applies to the Mini Dynamic Strut (CD209S, CD209SS, CD209M, CD209MS, CD209R, CD209RS) go to [www.coyote.us](http://www.coyote.us) for details.

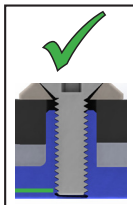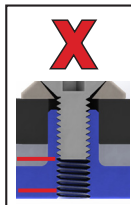

**CURBELL**  
PLASTICS

We typically use Curbell Plastic's - ProComp®  
Carbon Reinforced Polypropylene.

[www.coyote.us/procomp](http://www.coyote.us/procomp)



**See instructions online.**

[www.coyote.us/instructions-dynamicstrut](http://www.coyote.us/instructions-dynamicstrut)

EN | Instructions for Use  
DE | Gebrauchsanweisung  
FR | Notice d'utilisation  
ES | Instrucciones para el uso  
IT | Istruzioni per l'uso  
NO | Bruksanvisning  
NL | Gebruiksaanwijzing  
PT | Instruções de Utilização  
PL | Instrukcja użytkowania  
RU | Инструкция по использованию

CS | Návod k použití  
JA | 取扱説明書  
ZH | 中文说明书  
KO | 사용 설명서  
DA | Brugsanvisning  
SV | Bruksanvisning  
EL | Οδηγίες Χρήσης  
FI | Käyttöohjeet  
TR | Kullanım Talimatları

