

## Parts Included



Small Mix Tip (1)

### Tech Tip -

Store all glue in cool environment for longevity.

Coyote Quik Glue 50cc

EN | Instructions for Use  
DE | Gebrauchsanweisung  
FR | Notice d'utilisation  
ES | Instrucciones para el uso  
IT | Istruzioni per l'uso  
NO | Bruksanvisning  
DA | Brugsanvisning  
SV | Bruksanvisning  
EL | Οδηγίες Χρήσης  
FI | Käyttöohjeet  
NL | Gebruiksaanwijzing  
PT | Instruções de Utilização  
PL | Instrukcja użytkowania

CS | Návod k použití  
TR | Kullanım Talimatları  
RU | Инструкция по использованию  
JA | 取扱説明書  
ZH | 中文说明书  
KO | 사용 설명서



[www.coyote.us/instructions-quickglue](http://www.coyote.us/instructions-quickglue)



Manufactured by Coyote®  
419 N. Curtis Rd., Boise, Idaho 83706  
(208) 429-0026 | [www.coyote.us](http://www.coyote.us)



# QUIK GLUE

## TAN - CD4350

60 second set time - 50cc



Made in U.S.A.




CD4350.revD.02082023


8 11576 02235 4




## Instructions

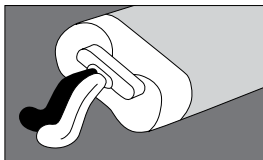


- 1** Purge the cartridge  
Snip off the tip. Insert cartridge into dispenser gun. Expel a small amount of adhesive to purge the cartridge.

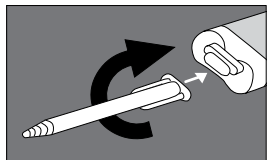

- 2** Install mix tip  
Line up mix tip appropriately and twist into place. Tips can only be used once.

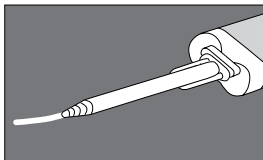

- 3** Run a test bead approx. 1" in length. Store unused portion by leaving the mix tip on when done.

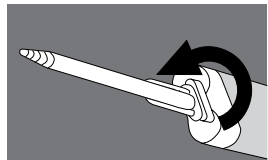


- 4** Leave used tip attached as a cap until next use to keep glue from drying out in the tubes. When your ready to use the glue again remove old tip and replace with a new tip.

## CAUTION

1. Read carefully BEFORE applying adhesive to ensure proper adhesion.
2. Quik Glue sets fast. To ensure a proper bond, work quickly. Refrigerate the cartridge to slow down work time.
3. Smaller bead size may affect cure time.


## Parts Sold Separately

**CDDMA51** | Dispenser is required for use (sold separately)

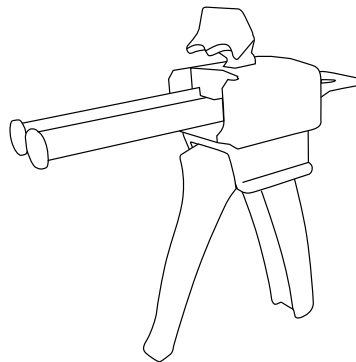

**CD401** | 6-Pack Extra Mix Tips for 50 cc Cartridge.

**CD402** | 12-Pack Extra Mix Tips for 50 cc Cartridge.

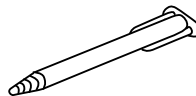

Contact Coyote for a MSDS for this product.  
(208) 429-0026 or [www.coyote@coyote.us](mailto:www.coyote@coyote.us)