

Parts Included



Coyote®

CD120L Lanyard and Low Profile Screw

Fabrication Instructions



2-year warranty against manufacturer defects, excessive wear or breakage.

External Prosthetic Components



EC REP

Advena Limited
Tower Business Centre
2nd Flr, Tower Street
Swatar, BKR 4013
Malta



CD120L.revC.01292024

EN | Instructions for Use
 DE | Gebrauchsanweisung
 FR | Notice d'utilisation
 ES | Instrucciones para el uso
 IT | Istruzioni per l'uso
 NO | Bruksanvisning
 DA | Brugsanvisning
 SV | Bruksanvisning
 EL | Οδηγίες Χρήσης
 FI | Käyttöohjeet
 NL | Gebruiksaanwijzing
 PT | Instruções de Utilização
 PL | Instrukcja użytkowania

CS | Návod k použití
 TR | Kullanım Talimatları
 RU | Инструкция по использованию
 JA | 取扱説明書
 ZH | 中文说明书
 KO | 사용 설명서




www.coyote.us/instructions-Lanyard-puck

Coyote®


Manufactured by Coyote®
419 N. Curtis Rd., Boise, Idaho 83706
(208) 429-0026 | www.coyote.us




Attaching Lanyard Strap

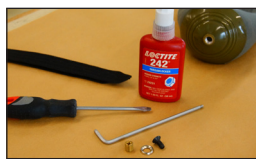

1 Attaching strap to liner.

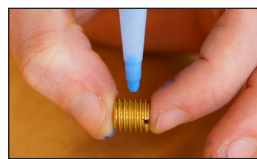

2 Use Loctite Blue 242 on all threads.

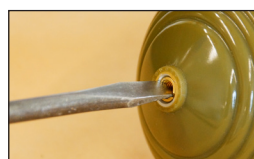

3 Screw adaptor screw in until it is flush with the liner.



4 Use a hot awl or punch to make a hole for 6x14 screw. *Screw should fit through hole easily.*



5 Put 6x14 screw and liner washer through hole in the lanyard strap.



6 Use Loctite 242 on 6x14 screw.




7 Screw in 6x14 Screw until it is tight.



8 Attach strap to liner and feed strap through lanyard slot.



9 Use lanyard strap to attach chafe in proper location.

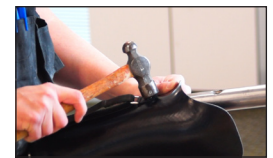

10 Add desired rivet to hold chafe.



11 Cut strap to desired length.



ATTENTION

Regardless of threading, always use Loctite® Blue 242 on lock pin threads. Follow liner manufacture instructions as they can vary.