




# CD207XR

## Extra Rigid Flex Kit

Dynamic Strut *made of Resilergy™*

Weight Range 260-350 lbs.

Light to Medium Activity Level

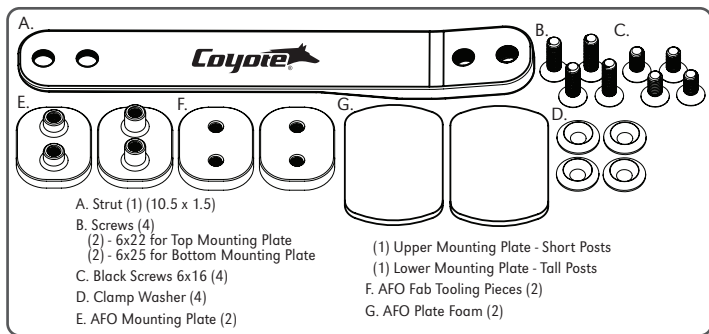

6-month warranty against manufacturer defects, excessive wear or breakage.

External Prosthetic Components



CD207XR.post.revA.03242025



EC REP

Manufactured by Coyote® O&P  
419 N. Curtis Rd., Boise, Idaho 83706 USA  
(208) 429-0026 | [www.coyote.us](http://www.coyote.us)



Advena Limited  
Tower Business Centre  
2nd Flr, Tower Street  
Swatara, BKR 4013  
Malta

## **ATTENTION**


**Always, *LOCTITE*® and torque to manufacturer specifications.**

1. The instructions for this device cover a very specific method of fabrication, and we understand that a multitude of methods could be used. The fabricated device is the sole responsibility of the fabricator.
2. Always use Loctite® Blue 242 on screw threads. Torque between 5 Nm and 8 Nm in even distribution.
3. Tighten the screws after a couple weeks of use, also at follow up appointments, re-Loctite® if needed.
4. Use the 6x20mm (top mounting plate) and 6x22mm (bottom mounting plate) screws provided with typical components for the (CD207XS, CD207XSS, CD207S, CD207SS, CD207M, CD207MS). Use the 6x22mm (top mounting plate) and 6x25mm (bottom mounting plate) screws provided with typical components for the (CD207R, CD207RS, CD207XR, CD207XRS).

In atypical setups, longer screws may be needed. Always use screws class 10.9 or better. Make sure the screws are all the way through the threaded post holes and flush with the inside of the plate, longer screws may be needed depending on the strut and brace thickness. See below example images full instruction booklet can be found online.

A 6 month limited warranty applies to the Dynamic Strut AFO (CD207XS, CD207XSS, CD207S, CD207SS, CD207M, CD207MS, CD207R, CD207RS, CD207XR, CD207XRS)

See instructions at [www.coyote.us/instructions](http://www.coyote.us/instructions) for more details.

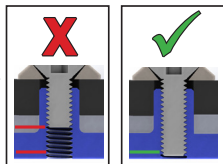

**CURBELL**  
PLASTICS

We typically use Curbell Plastic's - ProComp®  
Carbon Reinforced Polypropylene.

[www.coyote.us/procomp](http://www.coyote.us/procomp)



**See instructions online.**

[www.coyote.us/instructions-dynamicstrut](http://www.coyote.us/instructions-dynamicstrut)

EN | Instructions for Use

DE | Gebrauchsanweisung

FR | Notice d'utilisation

ES | Instrucciones para el uso

IT | Istruzioni per l'uso

NO | Bruksanvisning

NL | Gebruiksaanwijzing

PT | Instruções de Utilização

PL | Instrukcja użytkowania

RU | Инструкция по использованию

CS | Návod k použití

JA | 取扱説明書

ZH | 中文说明书

KO | 사용 설명서

DA | Brugsanvisning

SV | Bruksanvisning

EL | Οδηγίες Χρήσης

FI | Käyttöohjeet

TR | Kullanım Talimatları

