



Lanyard Strap

CD121L

Lanyard Spare Strap

Fabrication Instructions



Weight limit: 265 lbs.

2-year warranty against manufacturer defects, excessive wear or breakage.

Made in U.S.A.

External Prosthetic Components



www.coyote.us/instructions



CD121L.revB.10182022



EC REP

Advena Limited
Tower Business Centre
2nd Flr, Tower Street
Swatar, BKR 4013
Malta

Manufactured by Coyote®

419 N. Curtis Rd., Boise, Idaho 83706 USA
(208) 429-0026 | www.coyote.us




- EN | Instructions for Use
- DE | Gebrauchsanweisung
- FR | Notice d'utilisation
- ES | Instrucciones para el uso
- IT | Istruzioni per l'uso
- NO | Bruksanvisning
- DA | Brugsanvisning
- SV | Bruksanvisning
- EL | Οδηγίες Χρήσης
- FI | Käyttöohjeet
- NL | Gebruiksaanwijzing
- PT | Instruções de Utilização
- PL | Instrukcja użytkowania

- CS | Návod k použití
- TR | Kullanım Talimatları
- RU | Инструкция по использованию
- JA | 取扱説明書
- ZH | 中文说明书
- KO | 사용 설명서



www.coyote.us/instructions-Lanyard-puck


Attaching Lanyard Strap

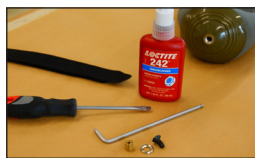

1 Attaching strap to liner.



2 Use Loctite 242 on all threads.

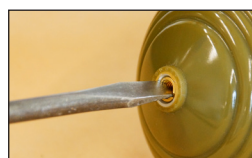

3 Screw adaptor screw in until it is flush with the liner.



4 Use a hot awl or punch to make a hole for 6x14 screw. Screw should fit through hole easily.



5 Put 6x14 screw and liner washer through hole in the lanyard strap.



6 Use Loctite 242 on 6x14 screw.




7 Screw in 6x14 Screw until it is tight.



8 Attach strap to liner and feed strap through lanyard slot.



9 Use lanyard strap to attach chafe in proper location.

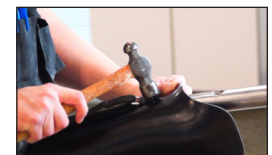

10 Add desired rivet to hold chafe.



11 Cut strap to desired length.