


# CD206FSP

## Flexion Shim Pair

Shims are designed to adjust planter flex or dorsiflex for  
Coyote<sup>®</sup> Dynamic Struts

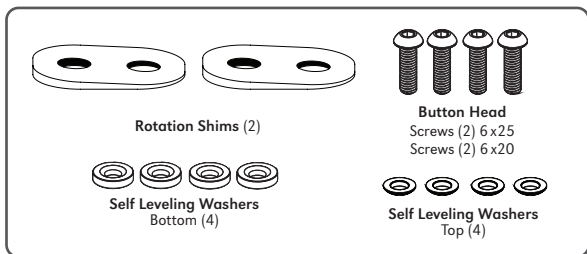

6 month warranty against manufacturer defects, excessive wear or breakage.  
External Prosthetic Components



CD206FSP.Post.revA.06272024



Manufactured by Coyote<sup>®</sup>  
419 N. Curtis Rd., Boise, Idaho 83706 USA  
(208) 429-0026 | [www.coyote.us](http://www.coyote.us)



Advena Limited  
Tower Business Centre  
2nd Flr, Tower Street  
Swatara, BKR 4013  
Malta

## Instructions

See Instruction example image on front.

### The Flexion shim is designed for planter flex and dorsiflex adjustment.

- 1 Install the shims between the AFO shell and the strut to adjust the foot shell or cuff in a planter flex or dorsiflex direction.
- 2 The shims are not designed to be stacked.
- 3 Make sure the screws are all the way

through all the threaded post holes and not sticking out past the inside of the mounting plate.

- 4 Make sure the washers are flush and seat flat with the strut.
- 5 When all adjustments are made take the brace apart and Loctite® the threads with blue 242 Loctite®. Torque 5-8 Nm.
- 6 Tighten the screws after a couple weeks of use, also at follow up appointments, re-Loctite® if needed. Always torque 5-8 Nm.

## ⚠ ATTENTION

Always, **LOCTITE®** and torque to manufacturer specifications.

1. The instructions for this device cover a very specific method of fabrication, and we understand that a multitude of methods could be used. The fabricated device is the sole responsibility of the fabricator.
2. Always use Loctite® Blue 242 on screw threads. Torque between 5 Nm and 8 Nm in even distribution.
3. Tighten the screws after a couple weeks of use, also at follow up appointments, re-Loctite® if needed.

4. Use the 6x18mm (top mounting plate) and 6x20mm (bottom mounting plate) screws provided with typical components for the (CD207XS, CD207XSS, CD207S, CD207SS) Use the 6x20mm (top mounting plate) and the 6x22mm (bottom mounting plate) screws provided with typical components for the (CD207M, CD207MS). Use the 6x22mm (top mounting plate) and 6x25mm (bottom mounting plate) screws provided with typical components for the (CD207R, CD207RS, CD207XR, CD207XRS).


In atypical setups, longer screws may be needed. Always use screws class 10.9 or better.

Make sure the screws are all the way through the threaded post holes and flush with the inside of the plate, longer screws may be needed depending on the strut and brace thickness.

See example images full instruction booklet can be found online.

A 6 month limited warranty applies to the Dynamic Strut AFO (CD207XS, CD207XSS, CD207S, CD207SS, CD207M, CD207MS, CD207R, CD207RS, CD207XR, CD207XRS)

See instructions at [www.coyote.us/instructions](http://www.coyote.us/instructions) for more details.

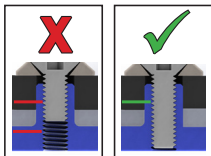

## See instructions online.

[www.coyote.us/instructions-dynamicstrut](http://www.coyote.us/instructions-dynamicstrut)

EN | Instructions for Use  
DE | Gebrauchsanweisung  
FR | Notice d'utilisation  
ES | Instrucciones para el uso  
IT | Istruzioni per l'uso  
NO | Bruksanvisning  
NL | Gebruiksaanwijzing  
PT | Instruções de Utilização  
PL | Instrukcja użytkowania  
RU | Инструкция по использованию

CS | Návod k použití  
JA | 取扱説明書  
ZH | 中文说明书  
KO | 사용 설명서  
DA | Brugsanvisning  
SV | Bruksanvisning  
EL | Οδηγίες Χρήσης  
FI | Käyttöohjeet  
TR | Kullanım Talimatları

