



Lanyard Strap

CD121L

Lanyard Spare Strap

Fabrication Instructions



Weight limit: 265 lbs.

2-year warranty against manufacturer defects, excessive wear or breakage.

- | | |
|--------------------------------|----------------------------------|
| EN Instructions for Use | CS Návod k použití |
| DE Gebrauchsanweisung | TR Kullanım Talimatları |
| FR Notice d'utilisation | RU Инструкция по использованию |
| ES Instrucciones para el uso | JA 取扱説明書 |
| IT Istruzioni per l'uso | ZH 中文说明书 |
| NO Bruksanvisning | KO 사용 설명서 |
| DA Brugsanvisning | |
| SV Bruksanvisning | |
| EL Οδηγίες Χρήσης | |
| FI Käyttöohjeet | |
| NL Gebruiksaanwijzing | |
| PT Instruções de Utilização | |
| PL Instrukcja użytkowania | |



www.coyote.us/instructions-Lanyard-puck

External Prosthetic Components



www.coyote.us/instructions

CD121L.revB.01302024



EC REP


Manufactured by Coyote®

419 N. Curtis Rd., Boise, Idaho 83706 USA
(208) 429-0026 | www.coyote.us




Advena Limited
Tower Business Centre
2nd Flr, Tower Street
Swatar, BKR 4013
Malta


Attaching Lanyard Strap

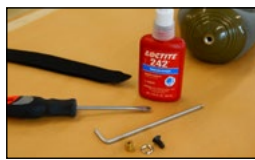

1 Attaching strap to liner.

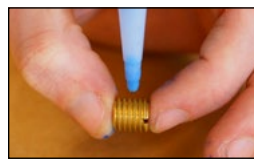

2 Use Loctite 242 on all threads.

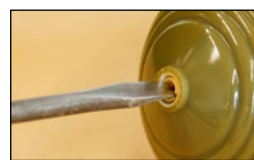

3 Screw adaptor screw in until it is flush with the liner.



4 Use a hot awl or punch to make a hole for 6x14 screw. Screw should fit through hole easily.



5 Put 6x14 screw and liner washer through hole in the lanyard strap.



6 Use Loctite 242 on 6x14 screw.




7 Screw in 6x14 Screw until it is tight.



8 Attach strap to liner and feed strap through lanyard slot.



9 Use lanyard strap to attach chafe in proper location.

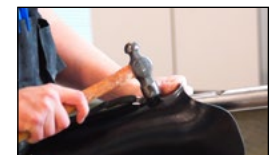

10 Add desired rivet to hold chafe.



11 Cut strap to desired length.



ATTENTION

Regardless of threading, always use Loctite® Blue 242 on lock pin threads. Follow liner manufacture instructions as they can vary.