



# CD207S

## Soft Flex Kit

Resilergy™ Dynamic Strut AFO

Weight Range 90-200 lbs.

Light to Medium Activity Level



6-month warranty against manufacturer defects, excessive wear or breakage.

External Prosthetic Components



[www.coyote.us/instructions](http://www.coyote.us/instructions)



CD207S.post.revA.08282023



EC REP

Manufactured by Coyote®  
419 N. Curtis Rd., Boise, Idaho 83706 USA  
(208) 429-0026 | [www.coyote.us](http://www.coyote.us)



Advena Limited  
Tower Business Centre  
2nd Flr, Tower Street  
Swatar, BKR 4013  
Malta

## Parts Included



- A. Strut (1) (10.5 x 1.5)  
B. Screws (4)  
    (2) - 6x18 for Top Mounting Plate  
    (2) - 6x20 for Bottom Mounting Plate  
C. Black Screws 6x14 (4)  
D. Clamp Washer (4)  
E. AFO Mounting Plate (2)  
    (1) Upper Mounting Plate - Short Posts  
    (1) Lower Mounting Plate - Tall Posts  
F. AFO Fab Tooling Pieces (2)  
G. AFO Plate Foam (2)

### ATTENTION

Make sure the screws are all the way through the threaded post holes and flush with the inside of the plate.


Use screws provided with the CD207S.

- (2) - 6x18 for Top Mounting Plate  
(2) - 6x20 for Bottom Mounting Plate

Always use screws class 10.9 or better.

Make sure the screw length fully seats into face plate not just the post, longer screws may be needed depending on the strut and brace thickness. Always use Loctite® Blue 242 on screw threads. Torque between 5 Nm and 8 Nm in even distribution. If the posts are sticking out past the face plate of the strut, remove the screws, strut, and grind post until it is flush with the strut surface.

See instructions at [www.coyote.us/instructions](http://www.coyote.us/instructions) for more details.

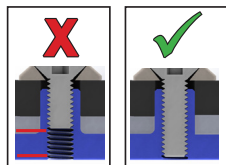

**CURBELL**  
PLASTICS

We typically use Curbell Plastic's - ProComp®  
Carbon Reinforced Polypropylene.

[www.coyote.us/procomp](http://www.coyote.us/procomp)



EN | Instructions for Use  
DE | Gebrauchsanweisung  
FR | Notice d'utilisation  
ES | Instrucciones para el uso  
IT | Istruzioni per l'uso  
NO | Bruksanvisning  
DA | Brugsanvisning  
SV | Bruksanvisning  
EL | Οδηγίες Χρήσης  
FI | Käyttöohjeet

NL | Gebruiksaanwijzing  
PT | Instruções de Utilização  
PL | Instrukcja użytkowania  
RU | Инструкция по использованию  
TR | Kullanım Talimatları  
CS | Návod k použití  
JA | 取扱説明書  
ZH | 中文说明书  
KO | 사용 설명서



[www.coyote.us/instructions-dynamicstrut](http://www.coyote.us/instructions-dynamicstrut)