

**Coyote** 

# CD207XRS

## Extra Rigid Flex Strut Only

Resilergy™ Dynamic Strut AFO

Weight Range 260-350 lbs.

Light to Medium Activity Level



[www.coyote.us/instructions](http://www.coyote.us/instructions)



6-month warranty against manufacturer defects, excessive wear or breakage. External Prosthetic Components



CD207XRS.post.revA.07202023



Manufactured by Coyote® O&P  
419 N. Curtis Rd., Boise, Idaho 83706 USA  
(208) 429-0026 | [www.coyote.us](http://www.coyote.us)



Advena Limited  
Tower Business Centre  
2nd Flr, Tower Street  
Swatar, BKR 4013  
Malta

## Parts Included



A. Strut (1) (10.5 x 1.5)

### ATTENTION

Make sure the screws are all the way through the threaded post holes and flush with the inside of the plate.

Use screws provided with the CD207R & CD207XR.


- (2) - M6x22 for Top Mounting Plate
- (2) - M6x25 for Bottom Mounting Plate
- (4) - M6x16 Black Screws (for tooling pieces)

Always use screws class 10.9 or better.

Make sure the screw length fully seats into face plate not just the post, longer screws may be needed depending on the strut and brace thickness.

Always use Loctite® Blue 242 on screw threads. Torque between 5 Nm and 8 Nm in even distribution. If the posts are sticking out past the face plate of the strut, remove the screws, strut, and grind post until it is flush with the strut surface.

See instructions at [www.coyote.us/instructions](http://www.coyote.us/instructions) for more details.

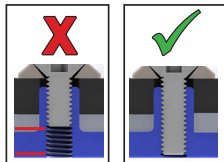

**CURBELL**  
PLASTICS

We typically use Curbell Plastic's - ProComp®  
Carbon Reinforced Polypropylene.

[www.coyote.us/procomp](http://www.coyote.us/procomp)



EN | Instructions for Use

DE | Gebrauchsanweisung

FR | Notice d'utilisation

ES | Instrucciones para el uso

IT | Istruzioni per l'uso

NO | Bruksanvisning

DA | Brugsanvisning

SV | Bruksanvisning

EL | Οδηγίες Χρήσης

FI | Käyttöohjeet

NL | Gebruiksaanwijzing

PT | Instruções de Utilização

PL | Instrukcja użytkowania

CS | Návod k použití

TR | Kullanım Talimatları

RU | Инструкция по использованию

JA | 取扱説明書

ZH | 中文说明书

KO | 사용 설명서



<https://www.coyote.us/instructions-dynamicstrut>