



# CD206FSP

## Flexion Shim Pair

Shims are designed to adjust planter flex or dorsiflex for  
Coyote® Dynamic Struts



[www.coyote.us/instructions](http://www.coyote.us/instructions)

CD206FSP.revA.10052022



Advena Limited  
Tower Business Centre  
2nd Flr, Tower Street  
Swatar, BKR 4013  
Malta

6 month warranty against manufacturer defects,  
excessive wear or breakage.

Manufactured by Coyote®  
419 N. Curtis Rd., Boise, Idaho 83706 USA  
(208) 429-0026 | [www.coyote.us](http://www.coyote.us)



## Parts Included



Flexion Shims (2)



### Button Head

A - Screws (2) 6 x25

B - Screws (2) 6 x20

### Self Leveling Washers


C - Bottom (2)

D - Top (2)

## Instructions

The Flexion shim is designed for planter flex and dorsiflex adjustment.

- 1 Install the shims between the AFO shell and the strut to adjust the foot shell or cuff in a planter flex or dorsiflex direction.
- 2 The shims are not designed to be stacked.
- 3 Make sure the screws are more than 3/4 of the way through all the threaded post holes and not sticking out past the inside of the mounting plate.
- 4 Make sure the washers are flush and seat flat with the strut.
- 5 When all adjustments are made take the brace apart and Loctite® the threads with blue 242 Loctite®. Torque 5-8 Nm.
- 6 Tighten the screws after a couple weeks of use, also at follow up appointments, re-Loctite® if needed. Always torque 5-8 Nm.

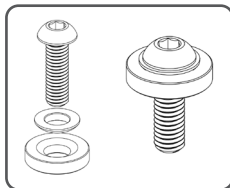

EN | Instructions for Use  
DE | Gebrauchsanweisung  
FR | Notice d'utilisation  
ES | Instrucciones para el uso  
IT | Istruzioni per l'uso  
NO | Bruksanvisning  
DA | Brugsanvisning  
SV | Bruksanvisning  
EL | Οδηγίες Χρήσης  
FI | Käyttöohjeet  
NL | Gebruiksaanwijzing  
PT | Instruções de Utilização  
PL | Instrukcja użytkowania

CS | Návod k použití  
TR | Kullanım Talimatları  
RU | Инструкция по использованию  
JA | 取扱説明書  
ZH | 中文说明书  
KO | 사용 설명서



[www.coyote.us/instructions](http://www.coyote.us/instructions)