

Coyote 

CD207RS

Rigid Flex Strut Only

Resilergy™ Dynamic Strut AFO

Weight Range 200-280 lbs.

Light to Medium Activity Level



www.coyote.us/instructions



6-month warranty against manufacturer defects, excessive wear or breakage. External Prosthetic Components



8 11576 02420 4

CD207R.post.revA.07182023



Manufactured by Coyote® O&P
419 N. Curtis Rd., Boise, Idaho 83706 USA
(208) 429-0026 | www.coyote.us



Advena Limited
Tower Business Centre
2nd Flr, Tower Street
Swatar, BKR 4013
Malta

Parts Included



A. Strut (1) (10.5 x 1.5)

ATTENTION

Make sure the screws are all the way through the threaded post holes and flush with the inside of the plate.

Use screws provided with the CD207R & CD207XR

(2) - M6x22 for Top Mounting Plate

(2) - M6x25 for Bottom Mounting Plate


(4) - M6x16 Black Screws (for tooling pieces)

Always use screws class 10.9 or better.

Make sure the screw length fully seats into face plate not just the post, longer screws may be needed depending on the strut and brace thickness.

Always use Loctite® Blue 242 on screw threads. Torque between 5 Nm and 8 Nm in even distribution. If the posts are sticking out past the face plate of the strut, remove the screws, strut, and grind post until it is flush with the strut surface.

See instructions at www.coyote.us/instructions for more details.

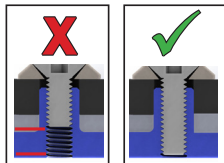

CURBELL
PLASTICS

We typically use Curbell Plastic's - ProComp®
Carbon Reinforced Polypropylene.

www.coyote.us/procomp



EN | Instructions for Use
DE | Gebrauchsanweisung
FR | Notice d'utilisation
ES | Instrucciones para el uso
IT | Istruzioni per l'uso
NO | Bruksanvisning
DA | Brugsanvisning
SV | Bruksanvisning
EL | Οδηγίες Χρήσης
FI | Käyttöohjeet
NL | Gebruiksaanwijzing

PT | Instruções de Utilização
PL | Instrukcja użytkowania
CS | Návod k použití
TR | Kullanım Talimatları
RU | Инструкция по использованию
JA | 取扱説明書
ZH | 中文说明书
KO | 사용 설명서



<https://www.coyote.us/instructions-dynamicstrut>