


Parts Included

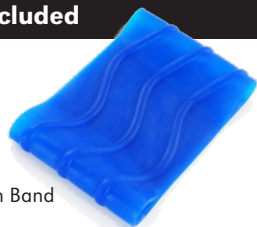

Wave Suction Band



2 oz. Silicone Adhesive

Wave Suction Band sizes

Part Number	Size	Circumference	Width	Thickness
CD130BS	Small	16cm - 25cm	7.62cm	4mm
CD130BM	Medium	25cm - 37cm	7.62cm	4mm
CD130BL	Large	37cm - 49cm	7.62cm	4mm
CD130BX	Extra Large	49cm - 64cm	7.62cm	4mm
CD130BXX	Extra Extra Large	64cm - 77cm	7.62cm	4mm

EN | Instructions for Use
 DE | Gebrauchsanweisung
 FR | Notice d'utilisation
 ES | Instrucciones para el uso
 IT | Istruzioni per l'uso
 NO | Bruksanvisning
 DA | Brugsanvisning
 SV | Bruksanvisning
 EL | Οδηγίες Χρήσης
 FI | Käyttöohjeet
 NL | Gebruiksaanwijzing
 PT | Instruções de Utilização

PL | Instrukcja użytkowania
 CS | Návod k použití
 TR | Kullanım Talimatları
 RU | Инструкция по использованию
 JA | 取扱説明書
 ZH | 中文说明书
 KO | 사용 설명서



www.coyote.us/instructions-waveband



Manufactured by Coyote®
 419 N. Curtis Rd., Boise, Idaho 83706
 (208) 429-0026 | www.coyote.us



CD130BM

Medium Wave Suction Band

Circumference 25cm - 37cm, Width 7.62cm, Thickness 4mm

Fabrication Instructions



Weight limit: 265 lbs.

2-year warranty against manufacturer defects, excessive wear or breakage.

External Prosthetic Components



EC REP

Advena Limited
 Tower Business Centre
 2nd Flr, Tower Street
 Swatar, BKR 4013
 Malta

Attaching Seal Band




1 With liner on patient position band in desired location.




2 Mark band location on distal and proximal sides of band.




3 For unconventional band position mark band in multiple locations.

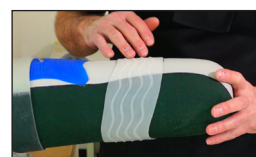


4 Wrap the liner with electrical tape or vinyl tape next to the band without overlapping it.

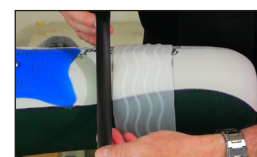

5 Tape off both ends of the band.

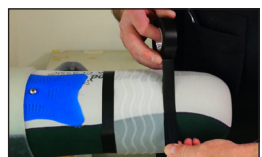


6 Remove the band.

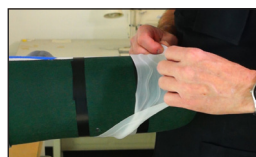


7 Add extra tape to protect a wider area.




8 Put a bead of silicone adhesive around the liner about the thickness of a pencil.

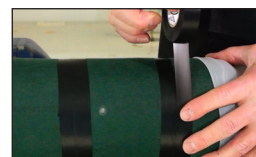

9 Smooth the silicone out with a tongue depressor. Make sure to saturate the material.

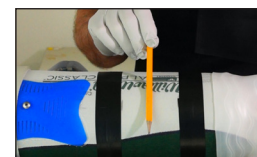


10 Check the edges for areas that need more silicone adhesive.

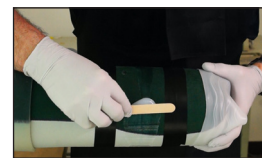


11 Smooth the silicone out with a tongue depressor. Make sure to saturate the material.




12 Carefully place the seal band over the glued area.

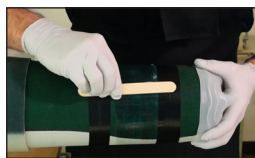


13 Center the band with tape to the edges.

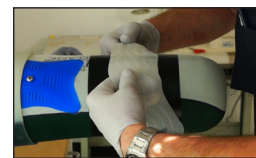


14 Smooth the silicone out.

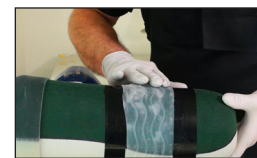


15 Smooth the silicone out.

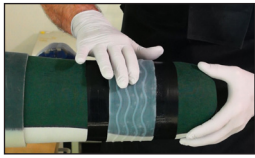


16 Push the air bubbles out from under the band

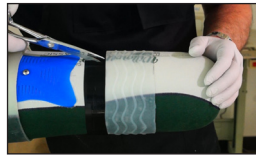


17 Remove the tape from the liner.

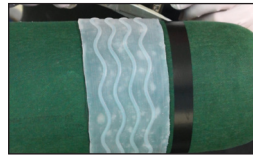

18 Create a seal band edge with silicone adhesive.

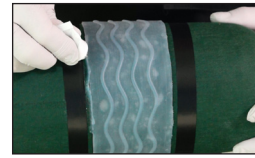


19 Roll tape next to band edge approximately 1/4 inch space from band.

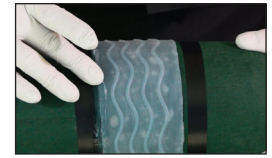

20 Smooth silicone into fabric and band. Making a sealed edge.



21 Remove tape.

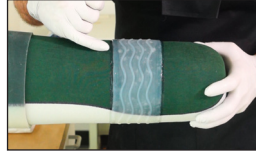

22 Check your work and let the silicone set up for at least two hours before putting it on patient.

Wave Suction Band sizes

Part Number	Size	Circumference	Width	Thickness
CD130BS	Small	16cm - 25cm	7.62cm	4mm
CD130BM	Medium	25cm - 37cm	7.62cm	4mm
CD130BL	Large	37cm - 49cm	7.62cm	4mm
CD130BX	Extra Large	49cm - 64cm	7.62cm	4mm
CD130BXX	Extra Extra Large	64cm - 77cm	7.62cm	4mm

Lay-up instructions are helpful hints on how to work with the lock and connector. Actual lay-ups are responsibility of the technician and/or practitioner.