



CD206RSP

Rotation Shim Pair

Shims designed to adjust lateral or medial rotation
for Coyote® Dynamic Struts



www.coyote.us/instructions

CD206RSP.revA.10052022



Advena Limited
Tower Business Centre
2nd Flr, Tower Street
Swatar, BKR 4013
Malta

6 month warranty against manufacturer defects,
excessive wear or breakage.

Manufactured by Coyote®
419 N. Curtis Rd., Boise, Idaho 83706 USA
(208) 429-0026 | www.coyote.us



Parts Included



Rotation Shims (2)



Button Head

A - Screws (2) 6 x25

B - Screws (2) 6 x20

Self Leveling Washers


C - Bottom (2)

D - Top (2)

Instructions

The Rotation Shim is designed for 2.5 degree lateral or medial rotational adjustments.

- 1 Install the shims between the AFO shell and the strut to adjust the foot shell or cuff's lateral or medial rotation.
- 2 The shims are not designed to be stacked.
- 3 Make sure the screws are more than 3/4 of the way through all the threaded post holes and not sticking out past the inside of the mounting plate.
- 4 Use the two piece adjustable washers and button head screws provided with the shims.
- 5 Make sure the large washers are flush and seat flat with the strut.
- 6 When all adjustments are made take the brace apart and Loctite® the threads with blue 242 Loctite®. Torque 5-8 Nm.
- 7 Tighten the screws after a couple weeks of use, also at follow up appointments, re-Loctite® if needed. Always torque 5-8 Nm.

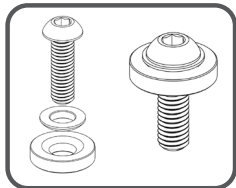

EN | Instructions for Use

DE | Gebrauchsanweisung

FR | Notice d'utilisation

ES | Instrucciones para el uso

IT | Istruzioni per l'uso

NO | Bruksanvisning

DA | Brugsanvisning

SV | Bruksanvisning

EL | Οδηγίες Χρήσης

FI | Käyttöohjeet

NL | Gebruiksaanwijzing

PT | Instruções de Utilização

PL | Instrukcja użytkowania

CS | Návod k použití

TR | Kullanım Talimatları

RU | Инструкция по использованию

JA | 取扱説明書

ZH | 中文说明书

KO | 사용 설명서



www.coyote.us/instructions