

Included in package:



1 Fabrication Dummy



# CD103SFD

## Fabrication Dummy

For CD103S, CD103SD

### Small Air-Lock and Small Deep Airlock



Manufactured by:



419 N. Curtis Rd., Boise, ID 83706 USA  
(208) 429-0026  
www.coyote.us

2-Year warranty against manufacturer defects, excessive wear or breakage

External Prosthetic Components



EC REP

Advena Limited  
Tower Business Centre  
2nd Flr, Tower Street  
Swatar, BKR 4013  
Malta



CD103FD.revA.03102015

Included in package:



1 Fabrication Dummy



# CD103SFD

## Fabrication Dummy

For CD103S, CD103SD

### Small Air-Lock and Small Deep Airlock



Manufactured by:



419 N. Curtis Rd., Boise, ID 83706 USA  
(208) 429-0026  
www.coyote.us

2-Year warranty against manufacturer defects, excessive wear or breakage

External Prosthetic Components



EC REP

Advena Limited  
Tower Business Centre  
2nd Flr, Tower Street  
Swatar, BKR 4013  
Malta



CD103FD.revA.03102015

## Flexible Inner Socket with Coyote Designs Fabrication Dummy CD103SFD



For Small Air-Lock put fabrication CD103SFD dummy over anchor.



Pull preferred flexible plastic over socket and installed fabrication dummy using a vacuum system.



Pressing in around fabrication dummy will help remove any air pockets.



If needed use parachute cord or rope to pull plastic tight to dummies edge.



Let plastic cool before cutting fabrication dummy free.



Cut tooling free with cast saw, knife or utility knife.



Remove tooling from anchor.



Sand flexible inner down so anchor is almost 100% exposed



Pull a PVA bag over the anchor and cast. Glue your lock to anchor and start laminating.

## Flexible Inner Socket with Coyote Designs Fabrication Dummy CD103SFD



For Small Air-Lock put fabrication CD103SFD dummy over anchor.



Pull preferred flexible plastic over socket and installed fabrication dummy using a vacuum system.



Pressing in around fabrication dummy will help remove any air pockets.



If needed use parachute cord or rope to pull plastic tight to dummies edge.



Let plastic cool before cutting fabrication dummy free.



Cut tooling free with cast saw, knife or utility knife.



Remove tooling from anchor.



Sand flexible inner down so anchor is almost 100% exposed



Pull a PVA bag over the anchor and cast. Glue your lock to anchor and start laminating.