

Coyote 

CD503T

Amputee Tool Kit



www.coyote.us/instructions

CD502T.revA.10262022





6 month warranty against manufacturer defects,
excessive wear or breakage.

Manufactured by Coyote®
419 N. Curtis Rd., Boise, Idaho 83706 USA
(208) 429-0026 | www.coyote.us



Parts Included

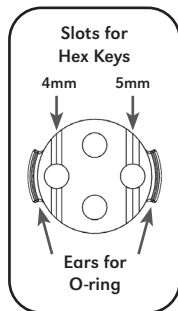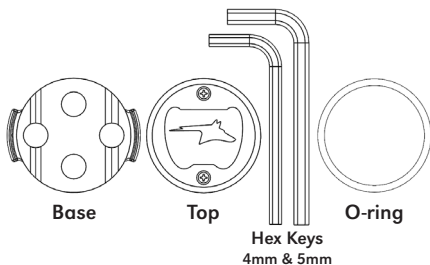

Instructions

- 1 Remove top from base. (*Top is held on by magnets*)
- 2 Put O-rings through side ears on base, wrap O-ring around pylon on prosthetic and wrap O-ring around ears on other side.
- 3 Put 4mm and 5mm hex keys in appropriate vertical slots and attach top with magnet to hold them in place.



ATTENTION

This product is for emergency adjustments only. If you need to adjust your prosthetic take it to a qualified technician or prosthetist for any needed adjustments. Be sure to use Loctite® Blue 242 to secure any loose screws and torque to manufacturer specifications.

EN | Instructions for Use
DE | Gebrauchsanweisung
FR | Notice d'utilisation
ES | Instrucciones para el uso
IT | Istruzioni per l'uso
NO | Bruksanvisning
DA | Brugsanvisning
SV | Bruksanvisning
EL | Οδηγίες Χρήσης
FI | Käyttöohjeet
NL | Gebruiksaanwijzing
PT | Instruções de Utilização
PL | Instrukcja użytkowania

CS | Návod k použití
TR | Kullanım Talimatları
RU | Инструкция по использованию
JA | 取扱説明書
ZH | 中文说明书
KO | 사용 설명서



www.coyote.us/instructions