

Coyote® 

PRODUCT CATALOG



ABOUT COYOTE

At Coyote we are not only concerned with making quality products and providing excellent customer service, but also with creating a quality place to work and live. Our efforts toward a friendly, environmentally responsible workplace have earned us a Blue Ribbon Small Business Award from the U.S. Chamber of Commerce.

Made in the USA

More than 90 percent of our products are made in the USA. Most are made within a few miles of our home office in Boise, Idaho.

Coyote

419 N. Curtis Rd. www.coyote.us
Boise, Idaho 83706 USA (208) 429-0026

Need more product info?

Visit us at www.coyote.us for more information, videos, tips and instructions.



TABLE OF CONTENTS


Warranty	1
Air-Lock.....	2
Easy-Off Lock.....	4
Grommet Lock	6
Drop-In Locks	7
Connectors.....	8
Accessories.....	10
Lanyard	11
Locking Lanyard	12
Proximal Lock	14
Summit Lock.....	16
Wave Suction Band.....	17
Dynamic Strut AFO	18
Small Pediatric AFO Strut ...	20
Adhesive	22
Valves	24
Integrator	25
Coyote Composite.....	26
Silent Air Compressor.....	29
Tools & Accessories.....	30
Kleenaire.....	32
Education.....	Back



2-Year Warranty

You want products to last and we want to learn how to make them last longer. This is why we offer a TWO YEAR warranty from date of purchase on most of our products* against manufacturer's defects, excessive wear, or breakage. You don't pay for shipping.**

- 1 Call and let us know if you need a part replaced. (208) 429-0026.
 - 2 We will need a PO to ship you a new part. We will then invoice you for the part and the shipping.
 - 3 We will fax or email you a Return Authorization Form for you to complete.
 - 4 Ship the damaged part and completed form to us. Charges bill to our UPS account.
 - 5 After inspection and approval*** of returned part, we will issue you a credit. No charge for ground shipping.
 - Return Authorizations will be rejected if the product is older than a year from the purchase date.
 - There is a 20% restocking fee for the return of any products after 6 months of purchase.
 - Any glue product older than 6 months will not be accepted.
 - ♦ This includes all Quik Glue products and all products that use silicone adhesive such as the Proximal Lock and the Wave Suction Band.
- * 6 month limited warranty against manufacturer defects, excessive wear, or breakage applies to the Dynamic Strut AFO (CD207XS, CD207S, CD207M, CD207MS, CD207SS, CD207XSS) Call for details.
- ** Parts must be used under reasonable and normal circumstances.
- *** All Kleenaire blowers are warranted for 1 year from date of original purchase. See warranty for details.

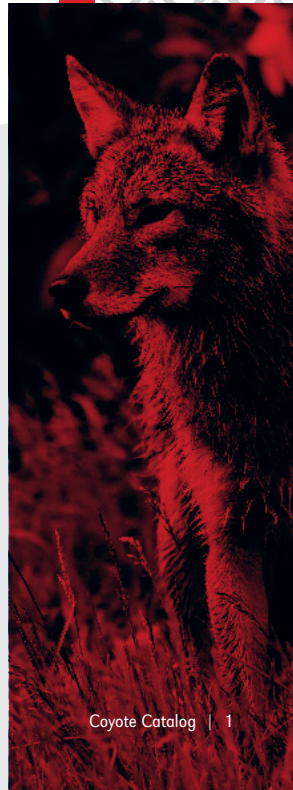

AIR-LOCK

MADE IN USA



A lightweight, dual-suspension pin system designed to combine suction suspension comfort with pin suspension security. The airtight seal prevents air from coming into the bottom of the socket, creating suction and reducing pistoning even without a sleeve!
For patients who struggle with pin engagement, use the Deep Air-Lock with the 11 click pin.

- ⊕ Suction allows for less traction on the limb for increased comfort and decreased liner wear
- ⊕ Expulsion valve or suspension sleeve are not needed to maintain suction in most cases
- ⊕ Water-resistant
- ⊕ Do not use with elevated vacuum



www.coyote.us/airlock



CD103WH
Lock wrench
Makes maintenance easy.

CD103 | Air-Lock

AK BK Patent No. 6334876

CD103M | Medium Air-Lock

AK BK AE BE PED

CD103S | Small Air-Lock

AK BK AE BE PED



Pins are now sold separately and come in a two pack.

Deep Air-Locks



CD103D - Deep Air-Lock



CD103MD - Medium Deep Air-Lock



CD103SD - Small Deep Air-Lock

ADDITIONS



CD103AF
Alignable Connector
Offset up to .6"



CD111
One-Shot Connector
Offset up to .6"



CD103SDI
Single Direction Insert
Offset up to .33"



CD103MDI
Multi-Direction Insert
Offset up to .33"



CD103P8
10 mm, Two Part
8-click Pin
350 lbs capacity
1.3" long. Includes spacers



CD103P11
10 mm, Two Part
11-click Pin
350 lbs capacity
1.65" long. Includes spacers



CD103P8H
10 mm, 8-click Brass
For active, heavier patients
350 lbs capacity
1.3" long. Includes spacers



CD103P11H
10 mm, 11-click Brass
For active, heavier patients
350 lbs capacity
1.65" long. Includes spacers



CD103PX
6 mm, Two Part
8-click Pin
1.3" long. Includes spacers



CD103PQ
1/4x20, Two Part
8-click Pin
1.3" long. Includes spacers

EASY-OFF™ LOCK

MADE IN USA



Our second generation lock has a more robust lever, tooth and larger O-ring. It is very similar to the Air-Lock in that it stays airtight to create suction in the socket, the Easy-Off™ Lock can be used with elevated vacuum. Its unique lever design also makes doffing easy, especially for amputees with hand strength issues, arthritis, or trouble reaching the release button.

- ⊕ Easy donning and doffing with lever release
- ⊕ Self-lubricating materials reduce wear
- ⊕ Water-resistant
- ⊕ Can be used with elevated vacuum
- ⊕ Airtight “suction” or vacuum suspension with the safety and security of a pin



CD117 | Easy-Off™ Lock

AK BK

Patent No. 6334876.
Other patents pending.



www.coyote.us/easyoff



CD117D | Deep Easy-Off™ Lock

AK BK

Easy removal without holding release button.
Push or pull to release.



Pins are now sold separately and come in a two pack.

ADDITIONS



CD103AF
Alignable Connector
Offset up to 0.6"



CD111
One-Shot Connector
Offset up to 0.6"



CD103SDI
Single-Direction Insert
Offset up to 0.33"



CD103MDI
Multi-Direction Insert
Offset up to 0.33"



CD103P8
10 mm, Two Part 8-click Pin
350 lbs capacity
1.3" long. Includes spacers



CD103P11
10 mm, Two Part 11-click Pin
350 lbs capacity
1.65" long. Includes spacers



CD103P8H
10 mm, 8-click Brass
For active, heavier patients
350 lbs capacity
1.3" long. Includes spacers



CD103P11H
10 mm, 11-click Brass
For active, heavier patients
350 lbs capacity
1.65" long. Includes spacers



CD103PX
6 mm, Two Part 8-click Pin
1.3" long. Includes spacers



CD103PQ
1/4x20, Two Part 8-click Pin
1.3" long. Includes spacers

GROMMET LOCK

Designed with pediatric patients in mind, the Grommet Lock is simple, lightweight, and durable for use with upper and lower extremity. Just like our adult locks, it has multiple connector options for fabrication of a lower extremity prosthetic.

CD104 | Grommet Lock

PED



- Ⓢ Pediatric only
- Ⓢ Lightweight, durable plastic
- Ⓢ Water-resistant
- Ⓢ Can be modified for minimum clearance
- Ⓢ Can be used in upper and lower extremity prosthetics
- Ⓢ Three thread options for any liner type (10mm, 6mm, ¼ x 20)
- Ⓢ Weight limit of 100 lbs



www.coyote.us/grommet

Pins are now sold separately and come in a two pack.



CD104P10 |
10mm Pin



CD104P6 |
6mm Pin



CD104PQ |
¼ x 20 Pin



CD104P6H |
6mm Pin



CD104PQH |
¼ x 20 Pin

ADDITIONS

DROP-IN LOCKS



CD122A | Air-Lock Drop-In

Build Height: 2.010"

First locks designed for 3D printed sockets and traditional fabrication.

- Ⓢ Water-Resistant
- Ⓢ Airtight suction - dual suspension security of a pin
- Ⓢ Expulsion valve or suspension sleeve not needed in most cases
- Ⓢ Great for 3D printed sockets

For 3D printing contact Coyote® for your STL and OBJ Files.



CD122E | Easy-Off Drop-In

2nd Generation - Build Height: 2.175"



Easy donning and doffing
with a more robust lever release and tooth

CD122AFD | Drop-In Air-Lock Fabrication Dummy

CD122AFD - Fabrication
dummy sold separately.



www.coyote.us/dropin



CD122EFD | Drop-In Easy-Off Fabrication Dummy 2nd Generation

CD122EFD - Fabrication
dummy sold separately.

Must use 2nd Generation
with the current lock design



Coyote Catalog | 7

CONNECTORS

MADE IN
USA



CD103AF | Alignable Connector

Up to 0.6" of offset, 35g. Ideal for use with Air-Lock and Easy Off Locks. Can be glued to the lock in any offset position for fabrication into the socket, or keys directly onto locks when used with inserts. Eliminates need for offset components. For laminations or thermoforming. 350 lb weight limit.




CD103MDI | Multi-Direction Insert

Up to 0.33" offset in two directions. Installed into connector to allow connector to be keyed onto bottom of lock in centered position or offset in two directions. For laminations or thermoforming. Use with Alignable Connector or One-Shot Connector.




CD103SDI | Single-Direction Insert

Up to 0.33" of offset in one direction. Installed into connector to allow connector to be keyed onto bottom of lock in centered position or offset in one direction. For laminations or thermoforming. Use with Alignable Connector or One-Shot Connector.

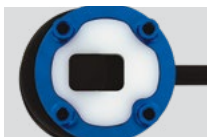

CD111 | One-Shot Connector

Up to 0.6" offset, 44g. Connector with tie off ring for easier lay-ups. Lower profile than Alignable Connector. Can be glued to lock in desired location or keyed onto lock using inserts. For laminating only. 350 lb weight limit.

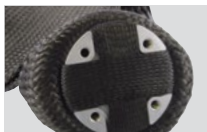

CD119SC | Test Socket Connector

Economical, cost effective, easy way to glue on connector designed specifically for Vivac check sockets. Connects to standard four hole componentry with our new high strength bridging design. Bonds well with Coyote Quik Adhesive. 11/16" height, 2 3/4" diameter, reusable.



www.coyote.us/connectors



CD106 | Alignment Coupler

Offset for ML and AP of 0.64". Easily installs on four hole connector. Easy to adjust with 3mm wrench. For fitting purposes only. 265 lb weight limit.



CD103PFF | Pediatric Fast Four Connector

Pediatric four-hole connector that keys directly onto the center of the lock for easy and fast fabrication. Used with Grommet Lock, Small Air-Lock or Medium Air-Lock. For laminations or thermoforming. 100 lb weight limit.



CD103PAF | Pediatric Alignable Connector

Used with Grommet Lock, Small Air-Lock or Medium Air-Lock. Laminating or thermoforming. This model can be used to offset the connector from the bottom of the pin in virtually any location by up to 0.25". 100 lb weight limit.



CD115CF5 | 5 Degree AK Connector

Capable of up to 1.5" of Offset with 5 degrees of flexion angle. Connectors primarily use is with Lanyard Pin (CD118P) lanyard locks (CD103L, CD117L) can also be used with other Coyote® locks or the Integrator. Eliminates the need for offset components. For lamination or thermoforming. 265 lb. weight limit.

ACCESSORIES

MADE IN USA



CD103FL | Fitting Lock

Air-Lock housing with reinforced distal end for accurate evaluation of lock pin spacing to improve seating of pin in lock. Ideal for reducing play in lock. Contact Coyote Design for long pin (Deep Air-Lock) fitting locks. *For fitting only.*



CD103GPN | Guide Pin

Used to make standard locks deeper. Prevents long pins from bottoming out in standard locks.

CD103RBC | Release Button Cover

Added to end of release button to make it easier for patients to release the lock. Ideal for geriatric patients or others with lower dexterity or limited hand strength.



CD103ELP | Exo Lamination Plug

Used when fabricating the Air-Lock in Exo sockets.

CD103PC | Pin Cap

The Pin Cap is used to reseal socket to keep it airtight when deepening socket when using a long pin with a standard lock.



LANYARD

MADE IN USA



www.coyote.us/Lanyard-puck

This is a simple, lightweight, ultra low build height lanyard lock that helps pistoning. Our new design for the metal puck has the connector posts built in for easier layups. The lanyard is segmented making easy length adjustments. They work well for simple visual donning and doffing and have K1 to K4 activity levels.

CD120 | Lanyard Plastic Puck

AK BK



CD121P | Lanyard Metal Puck with Posts

Integrated 4 hole connector

AK BK



Distal end of metal puck.

CD121D | Lanyard Aluminum Dummy with Posts



CD120L | Lanyard & low profile screw kit


24 inch strapping,
(puck not included)



CD121 | Lanyard Metal Puck

Integrated 4 hole connector

AK BK

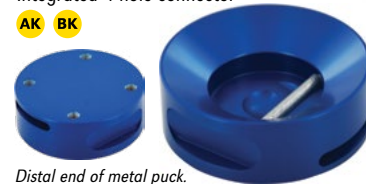

Distal end of metal puck.

LOCKING LANYARD

MADE IN
USA

Coyote®



- The lanyard pin helps elongate the limb when donning.
- The pin gives audible confidence of engagement.
- The o-ring at the end of the pin creates an airtight socket helping to reduce air noise.

Fabrication with the 5 Degree AK Connector eliminates the need for offset components.

Same simple fabrication as the Deep Air-Lock and Deep Easy-Off Lock pin systems.

ADDITIONS



CD115CF5 |
5 Degree AK Connector

Capable of up to 1.5" of Offset with 5 degrees of flexion angle.



CD118P |
Locking Lanyard Pin

350 lbs capacity, 1.65" long. Includes cord and spacers.



CD118PD |
Lanyard Pinch Disk

Holds Lanyard cord to side of socket.

CD117L | Easy-Off Locking Lanyard **AK**

2nd Generation, Robust Lever Design / Push Button, lanyard lock

- Easy donning and doffing with lever release
- Water-resistant
- Airtight "suction" with the safety and security of a pin



www.coyote.us/lanyard

CD103L | Air-Lock Locking Lanyard **AK**

Push Button Design, lanyard lock

- Suction allows for less traction on the limb for increased comfort and decreased liner wear
- Expulsion valve or suspension sleeve are not needed to maintain suction in most cases
- Water-resistant



One pin is included with the locking lanyard locks

PROXIMAL LOCK

MADE IN
USA

Simply easier. Designed to control rotation and be more user-friendly, the Proximal Lock attaches over a larger area, which distributes weight more evenly. The adhesive included with each lock is flexible even after it sets, making donning simpler.

CD116 | Proximal Lock

AK BK AE BE PED


Includes liner attachment, lock, fabrication dummy, glue form, and adhesive.

Patent No. 6666894, 7431738, 7077868.
Other patents pending.



The Proximal Lock includes a reinforcement strip for extra reinforcement, less friction and repair.

ADDITIONS

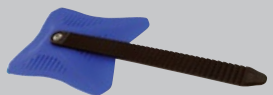

CD116XS | Extra Strap Kit

For additional liners. Includes strap and liner attachment.



CD116SA | Silicone Adhesive for Proximal Lock

Additional adhesive if needed. A tube is included with each lock.

- ZERO CLEARANCE lock
- Controls rotation
- Use with your choice of distal adapters
- Ideal for long limbs with minimal clearance
- Larger attachment distributes more tension
- Flexible attachment rolls on easier
- Can be used with seal band liners
- Adhesive included with each lock
- Lock can be retrofit
- Bills like a distal pin lock system (L5671)

Coyote®



www.coyote.us/proximal

SUMMIT™ LOCK MADE IN USA

Specifically for long limbs, the Summit™ Lock is designed to control rotation and requires zero clearance! There's no need to purchase a locking liner and you can use any socket adapter.

CD101 can be retrofit.



Base Length 1.75"

CD101 | Summit Lock

AK BK AE BE PED

Patent No. 6666894

www.coyote.us/summit

⊕ Liner attachment shape makes the liner simple to roll on

⊕ Easy to cover cosmetically

⊕ Ergonomic tab feeds easily into lock

CD102 | LowPro Summit™ Lock

AK BK AE BE PED

Patent No. 6666894



Base Length 2.5"



www.coyote.us/lowpro

Liner attachment is applied with "Coyote Quik Glue" or a 5 minute epoxy. Do not use silicone based adhesive.

ADDITIONS



CDDMA51 | Dispenser 50cc cartridge

Dispenser for CD4150 adhesive. Easy to control flow, light-weight.



CD401 | 6-Pack Extra Mix Tips

For 50cc cartridge

CD402 | 20-Pack Extra Mix Tips

For 50cc cartridge

WAVE SUCTION BAND

MADE IN USA

Wave Suction Band

The Coyote Wave Suction Band comes in multiple sizes.

- ⊕ The Suction Band helps create dual suspension in combination with our Proximal, Summit and Low-Pro Locks.
- ⊕ The airtight suspension produces a more intimate fit with added security.



www.coyote.us/wave-band

The Hands-Free Valve or Push Button Valve are recommended to relieve suction when donning and doffing.

Wave Suction Band sizes

Part Number	Size	Circumference	Width	Thickness
CD130BS	Small	16cm - 25cm	7.62cm	4mm
CD130BM	Medium	25cm - 37cm	7.62cm	4mm
CD130BL	Large	37cm - 49cm	7.62cm	4mm
CD130BX	Extra Large	49cm - 64cm	7.62cm	4mm
CD130BXX	Extra Extra Large	64cm - 77cm	7.62cm	4mm

Silicone adhesive is included.

The sizes are guidelines and can vary depending on the patient. Smaller sizes are more snug.



DYNAMIC STRUT AFO MADE IN USA

Our New Dynamic Strut Design combines a unique combination of strength and flex that other posterior dynamic struts do not offer. The AFO has the fit and comfort of a custom AFO with a posterior mounted dynamic strut for optimal ambulation. The strut is mounted to a custom thermoformed AFO and designed for average daily use; walking, working, hiking, biking, golf.

Has not been tested with running.

Designed to Flex

- Moves with the Patient
- Helps create a Natural Gate
- Works great with a thermoformed orthosis
- Offloads weight from the foot and ankle
- Provides Energy Response

Testimonials and demonstration videos can be found at www.coyote.us



www.coyote.us/dynamicstrut



CURBELL We typically make our dynamic strut PLASTICS AFO using Curbells ProComp™. This material gives us extra dynamic response in the toe while maintaining adjustability.
Exclusive ProComp™ Carbon Reinforced Polypropylene.

AFO Strut Choices

CD207M (Medium Flex Kit)


Patient Weight Range: 125 – 225 lbs
Patient Activity Level: Light to Medium Activity Level

CD207S (Soft Flex Kit)

Patient Weight Range: 90-200 lbs
Patient Activity Level: Light to Medium Activity

CD207XS (Extra Soft Flex Kit)

Patient Weight Range: 70-150 lbs
Patient Activity Level: Light to Medium Activity

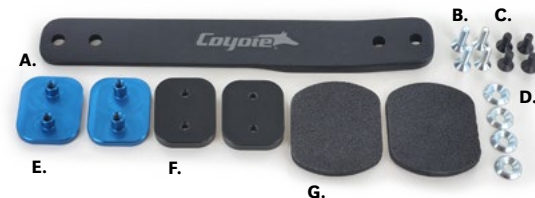

Replacement strut part numbers

CD207MS - Medium Flex Replacement Strut
CD207SS - Soft Flex Replacement Strut
CD207XSS - Super Soft Flex Replacement Strut

What comes in the package

- A. Strut (1) (10.5 x 1.5)
- B. Screws (4)
(2) - 6x18 for Top Mounting Plate
(2) - 6x20 for Bottom Mounting Plate
- C. Black Screws 6x14 (4)
- D. Clamp Washer (4)
- E. AFO Mounting Plate (2)
(1) Upper Mounting Plate - Short Posts
(1) Lower Mounting Plate - Tall Posts
- F. AFO Fabrication Tooling Pieces (2)
- G. AFO Plate Foam (2)



SMALL PEDIATRIC AFO STRUT

CD205PS | Small Pediatric AFO Strut

Patient Weight Range: up to 90 lbs.

Patient Activity Level: All Activity Levels

A more streamlined AFO that vents well.

Lighter weight than a solid AFO.

Practitioner Points

- Made from a composite plastic
- More torsion (Axial Rotation)
- Helps with hemiplegia, Cerebral Palsy, Spina Bifida when there is correction needed in the foot.
- Helps with Tri-planar control.
- Dynamic return with Plantar flexion Resist/ Dorsiflexion Resist.



MADE IN
USA

Coyote®

Technical Points

- Simple design
- Adds strength, reinforcement and support to Thermoplastic AFOs.
- All activity levels for children up to 90 lbs.
- Locking concave design for strength and thermoplastic vacuum drawing compatibility.



www.coyote.us/pediatric-strut



ADHESIVE

MADE IN
USA



Ideal Multi-Purpose Adhesive

Great for any situation when you need something glued quickly and easily. Both Adhesives ship non-hazardous and are safe with no odor.

- ⌚ Very quick set with no sag



CD4220 | Quik Glue White

30 Second, 220 cc cartridge. Dispenser required; includes 1 mix tip.

30 Second set up time
2 minute full cure time

CD4150 | Quik Glue White

30 Second, 50cc cartridge. Dispenser or plunger required; includes 1 mix tip.

NEW ADHESIVE



- ⌚ 60 second with no sag
- ⌚ Is solid when cured
- ⌚ Bonds well to fabric, plastic and metal
- ⌚ Easy to smooth

CD4320 | Quik Glue Tan

60 Second, 220 cc cartridge. Dispenser required; includes 1 mix tip.

60 Second set up time
3 minute full cure time

CD4350 | Quik Glue Tan

60 Second, 50cc cartridge. Dispenser or plunger required; includes 1 mix tip.

Stays Flexible when cured

- ⌚ Longer set time with no sag
- ⌚ Stays flexible when cured
- ⌚ Bonds well to fabric, plastic and metal
- ⌚ Easy to smooth



www.coyote.us/quikglue



CD4220S | Quik Glue Black

90 Second, 220 cc cartridge. Dispenser required; includes 1 mix tip.

90 Second set up time
7 minute full cure time

CD4250 | Quik Glue Black

90 Second, 50cc cartridge. Dispenser or plunger required; includes 1 mix tip.

*Recommend purchasing additional tips - Tips must be replaced prior to each use.

VALVES

MADE IN USA

Coyote's auto expulsion valves are easy to install in new sockets with fabrication tooling or retrofit. The built-in auto expulsion quietly expels air all day with no fuss and features a higher "crack" pressure than most valves, which helps to create a higher level of hypobaric suspension.

CD105 | CQL™ Auto Expulsion Valve

Height: 11mm. Patent pending.

- Easy to clean and retrofit
- Quiet expulsion
- Low profile
- Includes a filter to block debris



CD105HF | HandsFree™ Valve

Height: 8.5mm. Patent pending.

- Works with seal band liners
- Manual twist open airflow
- Flexible inner style valve



CD103WH | Lock wrench

Makes maintenance easy.



www.coyote.us/valves

THE INTEGRATOR™



www.coyote.us/integrator

The Integrator Socket Adapter, when used with one of our connectors, can be built into alignment in one step with a clean inside socket finish. Ideal for AK or BK sockets.

CD108 | Integrator™ (2 3/8" diameter) AK BK

Standard size, fits most lower extremity limbs. Integrator glues to mold. Alignable Connector with or without inserts is keyed onto Integrator and both parts are fabricated into socket. 265 lb weight limit. Thermoforming or laminating.



CD108S | Small Integrator™ (2" diameter) AK BK

Smaller size for smaller limb shapes. 265 lb weight limit. Thermoforming or laminating.

Put it to the Test

The Proximal lock, LowPro and Summit Locks were designed for Coyote® CEO and Co-Founder Matt Perkins.


Matt won 5 ITU Triathlon World Championships in 2006, 2007, 2008, 2009, 2010 while wearing the Proximal Lock.



COYOTE COMPOSITE

A great alternative to itchy, expensive carbon fiber braid! Coyote Composite, is made from basalt filament, a safe, tough material for composite sockets and AFOs. Also relatively easy to cut and finish compared to other materials.

- Extremely durable
- Resistant to cracking
- Less itch than carbon
- Non-carcinogenic and non-toxic
- Strong and lightweight
- Finishes smooth, with superior resin saturation
- Unique dampening characteristics
- Less expensive

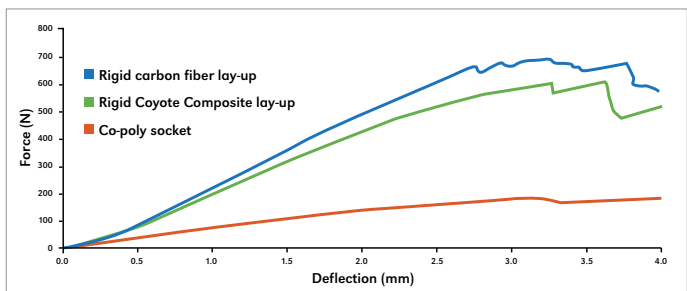

Resin Coyote Composite works with typical prosthetic resins and saturates much better than carbon.


Saturation and Laminating Because of its superior saturation it's important to prevent resin starvation. Lamination starvation will cause frayed edges.

Finishing Coyote Composite finishes easily. Use sharp tooling at slow speeds to finish. In some cases, you may need to wet-sand.

Relieving Coyote Composite is heat- and flame-resistant and isn't damaged by heating. It allows for limited adjustment based on the type of resin used.

Safety Because of the inert nature of basalt, it is not a carcinogen. Also, Coyote Composite is between 9 and 16 micrometers in diameter, which is too large to be an inhalant risk. Basalt also causes significantly less skin irritation.

Composite on outside of lay-up

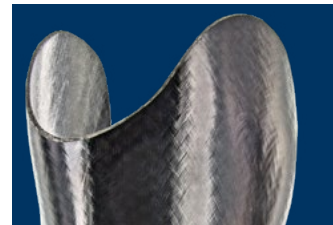


Composite on inside of lay-up



COYOTE COMPOSITE



Braid Our proprietary braid was designed to be used in equivalent amounts to carbon. Coyote Composite is more flexible than carbon (we consider this to be an advantage), but if you need to match the rigidity of a carbon socket, use additional layers of stockinette and/or carbon tape.

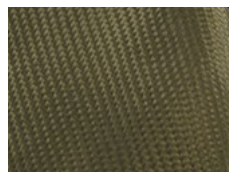

Fabric Coyote Composite fabric sheeting can be used for reinforcement layers of heavy duty sockets and/or AFOs. The material is thin and lightweight which allows for greater flexibility in adding the ideal amount of reinforcement.

Fabric	Type			Length/ (ft)
	Type	TPI*	Length/ (ft)	
CD	F	350	3	

Fabric available in
 .17 mm Thickness
 Has a Bi-directional weave

Ordering options

Braid	Type			Length/ (ft)
	Type	Size (in)	Length/ (ft)	
CD	B	3	10	
CD	B	3	20	
CD	B	3	82	
CD	B	4	10	
CD	B	4	20	
CD	B	4	82	
CD	B	5	10	
CD	B	5	20	
CD	B	5	82	
CD	B	6	10	
CD	B	6	20	
CD	B	6	82	
CD	B	7	10	
CD	B	7	20	
CD	B	7	82	
CD	B	8	10	
CD	B	8	20	
CD	B	8	82	
CD	B	10	10	
CD	B	10	20	
CD	B	10	82	

SILENT AIR COMPRESSOR



These silent air compressors are dependable, vibration free, and compact that have noise levels as low as 30 dB/A.



Each high performance compressor is built with quality in mind, and comes fully equipped with powder coated air tank, pressure switch, 5 micron line filter, pressure regulator, and gauges for easy installation and trouble free operation. When operating under full load this line of exceptionally quiet air compressors offers decibel levels as low as 30 dB/A.

Visit our website for more information of the different silent air compressor packages.



www.coyote.us/compressor

TOOLS & ACCESSORIES

MADE IN
USA



CD301 | Extractor Socket Removal Tool

The Extractor quickly and easily removes most molds from the laminated or thermoformed socket without damaging the mold or socket. Includes adapter for sockets with four-hole connectors.



CD307 | Plaster Spreader

Used for build-ups during modifications. The tool is easy to clean and won't stick to plaster.



www.coyote.us/tools



CD311 | 12 oz Mixing Bowl

Made of non-stick, durable material, its easy to grip and clean. Wide base won't tip over.

CD309 | Modification Handle

Allows for modifications while keeping fingers and knuckles clear of abuse. Easy to clean and replace blades, and reduces blade breakage. Works with 10" Stanley Surform Blades, flat- or half-round.



CD010 | Coyote Design Apron

Take care of your clothes in the lab with this inexpensive Coyote Design Smock. One size fits all.



CD103SFC | Small Casting Dots (200 pk)

CD103LFC | Large Casting Dots (50 pk)

Easily mark problem areas during the casting process. The adhesive back will keep them in place during casting. 0.375" or 0.87" diameter.



CD308 | Casting Bag

The Casting Bag, as seen in Q School workshops, is ideal for casting over to keep the liner clean. Sold in packs of 100.

CD317 | ML Gauge

This ML Gauge was designed specifically for measuring prosthetic and orthotic patients. Priced right and built to last.



Kleenaire

MADE IN
USA



New Price - New Design

KLEENAIRE | Dual Filtered Air Purification System

Patent US D536, 432S


- Ⓢ Reduces spread of viruses
- Ⓢ Lowers CO₂ levels
- Ⓢ Easy to install, maintain, and cost effective
- Ⓢ Controls dust, removes odors, contaminates, and even viruses
- Ⓢ Higher air quality and healthier working environment
- Ⓢ Designed for minimum noise and high air flow

The affordable alternative to purify the air for residential, commercial, or manufacturing



Clean air is now more important than ever
sign up for a filter change every quarter.

Coyote

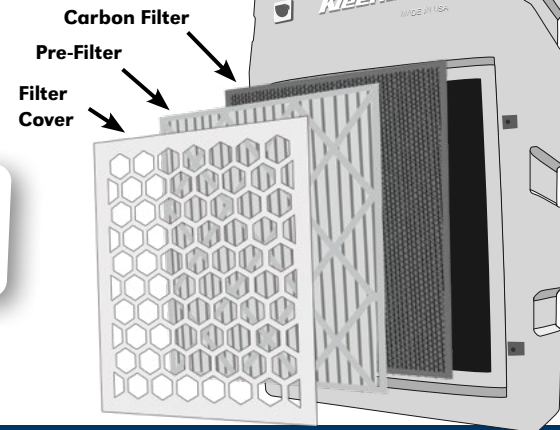

Buy your **filter subscription** at www.coyote.us/kleenaire/

See Manufacturer's Capacity Index chart for information on contaminants removed by the charcoal filter.

For more information go to www.coyote.us





Coyote has education opportunities everywhere

www.coyote.us • YouTube • opqschool.com • Webinars • Conferences

Tell your colleagues about our continuing education credits.



www.coyote.us



www.opqschool.com